AIR - FLOW® PLUS

Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Product Description

The AIR-FLOW® PLUS (prophylactic powder) is used for subgingival and supragingival treatments.

AIR-FLOW® Master and AIR-FLOW® Master Piezon users: this powder is neither compatible with the AIR-FLOW® nor with the PERIO-FLOW powder chambers. It has to be used only with the AIR-FLOW® PERIO+PLUS powder chamber.

This powder is for use in dentistry with the EMS air polishing devices.

Applications

This powder removes dental plaque and it is recommended for the following cleaning procedures:

SUBGINGIVAL applications:

- Removal of periodontal biofilm,
- Maintenance in periodontal therapy after completion of initial treatment,
- Implant cleaning.

SUPRAGINGIVAL applications:

- Removal of supragingival biofilm,
- Regular prophylaxis treatments and recalls,
- Removal of colorations around sulcus and in sensitive areas,
- Teeth polishing.

Contra-indications

- ⚠ Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

- ⚠ Predisposed persons may be sensitive to Chlorhexidin. If allergic reactions are observed, stop using the product and completely remove the powder from mouth.

- ⚠ The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients (endocarditis, pregnancy, breast feeding, contagious disease, immune deficiency (neutropenia, agranulocytosis, diabetes, hemophilia), patients under treatment (radiotherapy, chemotherapy, antitoxics), diabetes, hemophilia), patients under treatment (radiotherapy, chemotherapy, antitoxics).

- ⚠ Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

Recommendations for the treatment

- ⚠ Patient and user have to wear eye protection.

- ⚠ User has to wear a mask and gloves.

- ⚠ Use the high-speed evacuator of your dental unit.

- 👉 For the utilization of the PLUS powder with the AIR-FLOW® Master and AIR-FLOW® Master Piezon unit, only use the "AIR-FLOW® PERIO+PLUS" powder chamber.

- 👉 Calculus cannot be removed with this powder.

- 👉 Apply Vaseline® on the patient's lips before treatment

- 👉 Wash patient face after use (e.g. with a wet wipes).

General information

- Do not exceed the powder filling limit as indicated on the container.
- Do not stamp down the powder
- This powder is sugar free.

Storage and shelf-life

- The powder should not be stored near heat sources.
- Storage condition -10°C to 25°C and 10% to 90% relative humidity.
- Do not use after expiration date.
- The powder should not be stored more than one week inside the powder chamber.

Also available:

- AIR-FLOW® CLASSIC available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral (for supragingival treatments).
- AIR-FLOW® CLASSIC COMFORT (for supragingival treatments).
- AIR-FLOW® SOFT (salt free for supragingival treatments).
- AIR-FLOW® PERIO (salt free for subgingival application).

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