CE 0476

Mode d'emploi

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CE 0476

Instructions For Use

ENDO N2 is a root canal cement to be used exclusively by dentists.

Composition:

Table with 2 columns: Component and Amount. Lists ingredients like Zinc Oxide, Basic Nitrate Bismuth, etc.

Contraindications:

The use of ENDO N2 is contraindicated in patients showing an assessed hypersensitivity to the product.

Warning:

ENDO N2 is harmful if inhaled or swallowed or upon contact with the skin. Irritates eyes, skin, respiratory tract and the mucosa.

Dosage and directions for use:

Pour 3-4 drops of liquid ENDO N2 on a glass slide. Add small quantities of, ENDO N2 Powder until the 3-4 drops of liquid are absorbed.

Canal Filling Technique:

After carefully reaming the root canal, fill the root canal using a spiral, having previously chosen the adequate size, as follows: the spiral, loaded with a small quantity of ENDO N2 paste, is inserted into the canal, without rotating it, until it reaches the deepest level of the canal preparation.

Notes:

- As the canal has a minimum volume, it is, therefore, recommended to load the spiral each time with small quantities of ENDO N2. An exaggerated amount of the paste could block access to the canal and as such prevent the air inside the canal from escaping.

Notes for correct preservation:

Should the wrapping be damaged or show signs of spilling, use safety gloves to remove it. This type of waste is to be considered special. Do not use ENDO N2 beyond the expiration date printed on the package.

Appearance:

The package contains: 1 bottle contains 10 g of powder, 1 bottle contains 6 g liquid and 1 medicine dropper. The instructions for use were drawn up on 20/05/2015

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Table with 2 columns: Ingredient and Amount. Lists ingredients like Zinc Oxide, Basic Nitrate Bismuth, etc.

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Prodej: Hager & Werken GmbH & Co.KG, Ackerstraße 1 47269 Duisburg, Německo