VOCO

VOCO Profluorid Varnish

GB**CE** 0482

Fluoride-containing dental desensitising varnish

Instructions for use

Product description:

VOCO Profluorid Varnish is a fluoride-containing dental desensitising varnish for surface application to enamel and dentine.

VOCO Profluorid Varnish adheres to moist surfaces and is tolerant to moisture and saliva.

VOCO Profluorid Varnish is an ethanolic suspension of colophony with artificial flavouring and xylitol sweetening.

1 ml of **VOCO Profluorid Varnish** contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Indications:

VOCO Profluorid Varnish is intended for use for:

1. treatment of hypersensitive teeth
2. sealing the dentinal tubules, e.g. for cavity preparation or on sensitive root surfaces

Methods of use:

Recommended dosage

- **Treatment of patients with primary dentition**
approx. 0.25 ml of **VOCO Profluorid Varnish**
- **Treatment of patients with adolescent or permanent dentition** approx. 0.40 ml of **VOCO Profluorid Varnish**

VOCO Profluorid Varnish is applied as a thin film on the dental hard substance. Thick films peel off easily and do not provide therapeutic advantages.

By experience, the required amount for a complete set of teeth ranges between 0.25 ml and 0.40 ml.

Application:

1. Thoroughly clean tartar from treatment areas.
2. To improve results, eliminate any excessive moisture/saliva from area to be treated.
3. Uniformly apply **VOCO Profluorid Varnish** as a thin film, covering the entire target surface.
4. Moist area of application (either by gentle rinsing or natural salivary flow) to ensure proper setting of **VOCO Profluorid Varnish**.
5. Instruct patient to avoid solids, alcohol, brushing and flossing for 4 hours after application.

Information/precautionary measures:

VOCO Profluorid Varnish contains colophony.

VOCO Profluorid Varnish should not be used in case of known hypersensitivities (allergies) to this ingredient.

Colophony may inhibit polymerisation and adhesion of composites.

The cartridge must always be used with the blunt application cannulae intended for that purpose (length 32 mm, diameter 0.9 mm). **VOCO Profluorid Varnish** cartridges are reusable. Remove cannula after application and store cartridge without cannula (cartridge is sealed tightly by the rubber septum). Attach a new cannula for the next application.

According to its purpose, **VOCO Profluorid Varnish** has a sticky consistency. Material remaining on the screw thread of the tube may cause the cap to stick. Wipe off any contaminating residues with an ethanol-containing cloth.

VOCO Profluorid Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis.

VOCO Profluorid Varnish is not intended for systemic treatment.

Oedematous swellings have been reported on rare occasions especially following application to extensive surfaces.

Dyspnoea in asthmatic children has been reported for comparable preparations in extremely rare cases.

Patients with sensitive digestive systems may experience nausea, usually after extensive treatment.

In case of hypersensitivity, remove varnish with a soft toothbrush and floss, then rinse with warm water.

No application of other fluoride preparations should be carried out on day of treatment.

Suspend treatment with fluoride tablets for several days after treatment with **VOCO Profluorid Varnish**.

Do not swallow the varnish.

Storage:

Store at temperatures between 4 °C - 23 °C. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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