GC Tooth Mousse

TOPICAL CREME WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE

For use by and/or the supervision of a dental professional in the recommended indications.

**Description**
GC Tooth Mousse is a water based creme containing RECALDENT™ (CPP-ACP; Casein Phosphopeptide – Amorphous Calcium Phosphate). When applied to the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium and phosphate.

**Important:**
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

**A topical paste containing bio-available calcium and phosphate**
1. Provides extra protection for teeth.
2. Helps neutralise bacteria from acidogenic bacteria in plaque.
3. Helps neutralise acid challenges from other internal and external acid sources.

**CONTRAINDICATIONS**
RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use the product on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

**DIRECTIONS FOR USE**
**INDICATIONS**
Prior to any following in-office bleaching
a. After ultrasonic, hand scaling or root planing
c. Following professional tooth cleaning (P.T.C.).
d. Following orthodontic treatment.
e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
f. During orthodontic treatment.
g. For high risk children with orthodontic appliances.
h. For special needs patients.

**In-office application:**
**I. Custom tray application**
1. Before use, the custom tray thoroughly under running water.
2. Extrude a generous layer of GC Tooth Mousse into the tray and apply to the upper and / or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to avoid eating or drinking for as long as possible (1–2 minutes).
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual GC Tooth Mousse in the tray should be rinsed or brushed off under running water immediately after use.

**II. Non tray application:**
1. If necessary, remove any excess saliva on the tooth surface with a cotton swab. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using an application brush. Avoid spreading the product on adjacent areas using an Interproximal Tooth Cleaning Brush.
3. Leave GC Tooth Mousse undisturbed for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1 – 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

**At home application:**
**III. Day time application after tooth brushing as recommended by a dental professional**

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required.
2. Hold the material against the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

**IV. Night time application after tooth brushing as recommended by a dental professional**

1. Apply a sufficient amount of GC Tooth Mousse to the upper and lower teeth. A pea size amount for each arch is the minimum required.
2. Hold the material against the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

**FLAVOURS**
Strawbery (S), Melon (M), Vanilia (V), Mint (I), Tutti-Fruiti (T)

Note: S, M, V, I and T in parentheses are abbreviations for each flavour. These abbreviations are marked on the tubes after lot numbers to show the flavour of the contents.
For example, 021003S.....Strawbery

**STORAGE**
Store in a cool dry low humidity area away from direct sunlight (8-25°C /46.4-77.0°F).
(Shelf life: 2 years from date of manufacture)

**PACKAGES**
Tubs containing 40g (35mL)
1. Assorted Pack
10 x 40g (2 each of Strawberry, Melon, Vanilia, Mint, Tutti-Fruiti)
2. Single Flavour Package
Each Flavour
Vanilia, Strawberry, Melon, Tutti-Fruiti, Mint
3. Refill Package
1 x 40g tube in each flavour

**NOTE:**
GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia. RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

**CAUTION**
1. Always replace cap after use and remove any residual paste or mousse around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product or to the preservative. In this event, discontinue the use of the product and refer to a physician

**REGISTRATION**
GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia. RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

**CAUTION**
1. Always replace cap after use and remove any residual paste or mousse around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product or to the preservative. In this event, discontinue the use of the product and refer to a physician

**REGISTRATION**
GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia. RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

**CAUTION**
1. Always replace cap after use and remove any residual paste or mousse around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product or to the preservative. In this event, discontinue the use of the product and refer to a physician

**REGISTRATION**
GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia. RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

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