# VOCO

## V-Posil Adhesive

### Instructions for use

#### Product description/Indication:

Adhesive for **VOCO** addition-curing impression materials.  
Adheres to metals and acrylics.

#### Instructions for use:

Apply a thin coat of **V-Posil Adhesive** onto the impression tray and allow it to dry for approx. 5 minutes. Warmth and compressed air curtail the drying time. When using plastic impression trays, please check that they are compatible with the adhesive.

Replace the cap on the bottle immediately after use!

After mixing the impression material, load it into the tray within 30 minutes.

Remains of impression material can be cleaned off the tray with cleaning solvent, petroleum ether or orange oil.

#### Warnings:

Do not use **V-Posil Adhesive** with condensation-curing silicones, polyether or polysulfide materials.

#### Precautionary measures:

Do not use after the expiration date.

Do not swallow the material! If accidentally swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with skin.

Do not allow the material to enter the sewer or water system to avoid environmental hazard. Do not dispose of with household waste.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Keep away from sources of ignition.

#### Side effects:

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases.

Do not use the material in case of known allergy to one of the ingredients.

#### Composition:

Silicone resin, solvent (ethyl acetate, xylene).

#### Storage:

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

#### Note:

Not all **VOCO** products are available in all countries.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. **VOCO** recognizes its responsibility to replace products if proven to be defective. **VOCO** does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

**CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.**

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.**

**For dental use only.**

An explanation of the symbols used in labeling can be found at [www.voco.com/symbols](http://www.voco.com/symbols)

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**VOCO GmbH**