NSK

Prophylaxis Powder FLASH pearl

Operation Manual

 The EU directive 93/42/EEC was applied in the design and production of this medical device.

OM-DZ0943MA 000

ENGLISH

Thank you for purchasing NSK's FLASH pearl air polishing powder. Make sure you read this product usermanual carefully before use, and keep it in a place where it can be referred to at any time.

INTENDED USE

This product is for use by dentists, therapists and hygienists only.

WARNING

- Do not use on the following patients
 - Those with respiratory diseases.
 - Those with pathologically deep periodontal pockets (6mm or more) or mucosal lesion.
 - Those with grave ulcers in digestive organs.
 - Those with hepatic dysfunction.
 - Those with lung or cardiac disturbance or ventilatory/respiratory impairment
 - Those with wounds, injuries or abnormality in the oral cavity.
 - Those with hyperemia, hemorrhage, or inflammation in the oral cavity.
 - Those who tend to develop an inflammation or sores in the oral mucous membrane.
 - Those with allergies.
 - Those wearing contact lenses.
 - Depending on the patient's constitution, allergy symptoms or stomatitis may be caused in very rare cases. If such symptoms are detected, stop using the product immediately.
- Never direct the powder jet directly on to the cementum, on the roots of teeth, decalcified enamel, filling materials, margins of prosthetic appliances and filling materials, crowns, etc.

CAUTION

- In case any powder gets in the eyes wash out immediately with lots of water and consult an ophthalmologist.
- Be very careful to avoid cleaning powder entering anybody's mucosal areas (eyes, nose etc) other than the oral cavity. Also, protect the face with a towel, etc., or protective glasses, etc., to prevent entry of cleaning powder into eyes.
- During use, both the operator and the assistant should always wear protective glasses and protective face masks.
- If the patient wears contact lenses, the contact lenses should be taken out before treatment.
- Make sure to use sufficient suction to prevent patient from ingesting powder. Avoid long continuous use without allowing the patient to rinse. Always allow the patient to rinse regularly at appropriate times.
- Use the product in conjunction with spray water.
- Apply Vaseline, etc., to the patient's lips to prevent drying or chapping.
- Fill powder chamber with one sachet of FLASH pearl at a time. Discard any remaining powder in the powder chamber after use to avoid blockages.
- FLASH pearl is almost insoluble in water. To avoid clogging of the pipes by using FLASH pearl, regular cleaning and maintenance of suction hoses, saliva ejector and separator is necessary.


SPECIFICATIONS

- | | |
|--------------------------|-------------------|
| • Primary components | Calcium carbonate |
| • Mean particle diameter | 50~70µm |
| • Color | White grey |
| • Form | Spherical |

OPERATION

※ Follow the instructions in the Operation Manual of the Prophylaxis neo air polisher .

- Connect the instrument to the handpiece tubing .
- Open one sachet of cleaning powder and fill into powder chamber.
 - When using FLASH pearl powder in bottles open the powder chamber and fill up to the level indicated in the above diagram (just below the end

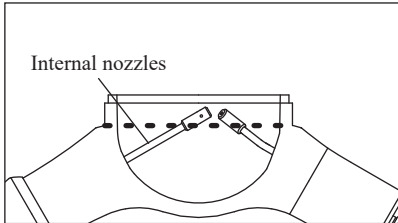of the internal nozzle). Firmly tighten the FLASH pearl bottle cap and the Prophylaxis neo powder chamber to avoid moisture getting to the powder.

- After filling the powder chamber, operate the Prophylaxis neo over the spittoon for 2 to 3 seconds to ensure good flow of powder and water before using it on the patient.
- Use the nozzle 3 to 5mm away from the tooth surface at angles of 10 to 60 degrees facing away from the gingiva.
- After use open the powder chamber and dispose of remaining powder to avoid contamination (moisture getting to the powder).

STORAGE

- NSK FLASH pearl should be stored in a cool and dry place. Protect from heat (>60°C).
- Do not use after the expiry date.

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